



Urine Collections & Transportation

- Tiger Top (red/yellow) Transport Tube is for Urinalysis (UA) only. Yellow Top Transport Tube is for all other urine tests, EXCEPT for Cytology samples.
- In addition to urinalysis and/or urine culture, Send Samples in the Following: Clear Cap No Additive Tube, Yellow Top Transport Tube or Sterile Cup (see below). Refer to www.iuhealth.net/pathologylab for additional information.
- If only a UA is ordered, send only the UA Transport Tube and/or Culture Tube. Discard the remaining sample.
- 24 Hr. Collections need to be measured with Total Volume recorded, and Two (2) Aliquots transferred into Sterile Cups. Be sure that the Lid is placed on correctly and screwed down tightly.



- 24 Hr. urines are to be measured into two aliquots (100 ml) submitted in 2 sterile containers
- Record the Total Volume and Collection Start and End times on the urine cup. Please bag Individually.
- Be sure to screw the lid on as tightly as possible. Urine cups that are not sealed properly will leak.
- See One-line Test Directory at www.iuhealth.net/pathologylab for individual test specifics
- Use the Urine Jug for 24 Hr. collections. Do NOT use water bottles, bleach, juice or milk bottles. They may be rejected in not in the correct container

Urine Culture Transport



UA Only Transport



Yellow Top Transport



Sterile Cup



The sterile container features a Quick Turn™ lid for leak proof transport and storage of urine and other liquid specimens

Clear Cap No Additive Transport

